


# Tactical Casualty Care Simulator (TCCS) Male


# QUICK START GUIDE

To get started, please follow these easy steps:


## 1 Equipment Identification

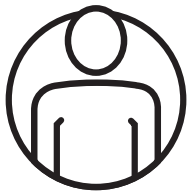


+ Simulator

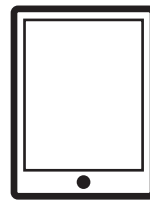

+ Instructor Tablet

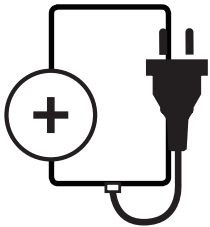

+ Battery Pack and Charger

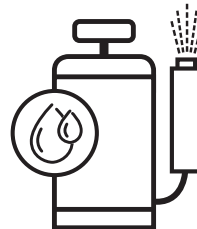

+ Hand Pump (for blood\*)



+ 2-Way Communication

\*Simulated blood also supplied.


## 2 Download the OEI App onto the Tablet

- + Turn on the tablet and select an available WiFi network
- + Download the OEI Sim application from the iOS or iPadOS App store, Mac OS App store, Google Play store, or Microsoft store
- + Go to the tablet's WiFi settings and disconnect the WiFi network you used to download the application



## 3 Charge Battery

- + Plug the battery pack charger into a standard electrical outlet. The red light indicates charging and turns green when the batteries are fully charged.
- + Charge the batteries six (6) to eight (8) hours (or overnight).

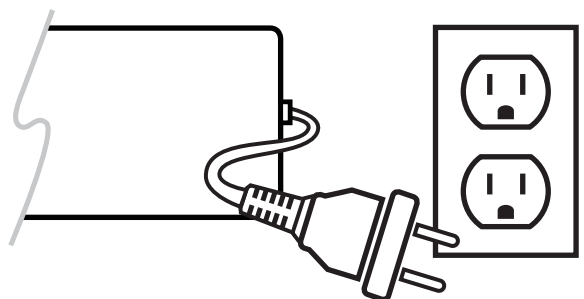


# TCCS Male QUICK START GUIDE

---

## 4 Add Blood to the Simulator

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- + Fill the reservoir on the hand pump with simulated blood.
- + Locate the blood fill port on the simulator's right waist.
- + Attach the hand pump's blood fill hose by snapping onto the blood fill port. (Be sure the hose nozzle is free of debris.)
- + Retract the handle and pump approximately three (3) times or until the simulator's blood reservoir is full.



## 5 Install Battery

- + Remove the cover on the upper left thigh to access and open the compartment.
- + Attach the battery to the connector.
- + Reset the battery in the compartment.
- + Replace the cover.



## 6 Power On

- + Press the Power button located in the battery console so that the light illuminates.
- + If using the Power Cord, plug the cord into the connection BE~~N~~DFOUUP the power button, securing by turning clockwise to lock into place.
- + Allow the simulator approximately two (2) minutes to cycle through the powering up sequence.



# TCCS Male QUICK START GUIDE

## 7 Connect to the Simulator's WiFi Network



- + Go to the instructor tablet's wireless access.
- + Locate the WiFi Network indicated by identification and serial number
- + Enter the Password: **oeiadmin**


## 8 Access the OEI SIM App on the Tablet



- + Click the shortcut icon on the Instructor Tablet to open the OEI Sim application. The software loads and the Welcome screen appears.

## 9 Select the Simulator


- + Locate the simulator from the list of **Available Simulators**.
  - + **Click Connect.**
- The application connects to the simulator and the Instructor Interface appears.

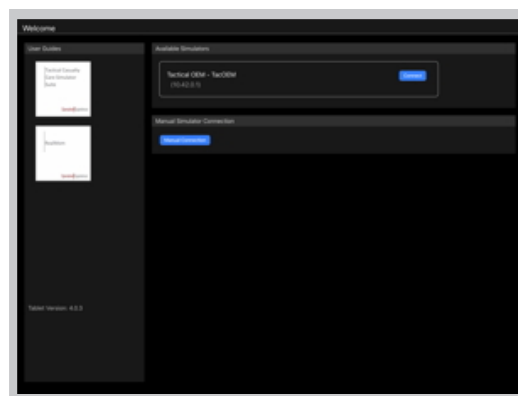

## 10 Begin Simulation

- + Press the **Play** button on the Instructor Interface to begin the simulation.


See full user manual at [operativeexperience.com/support](http://operativeexperience.com/support) for detailed instructions

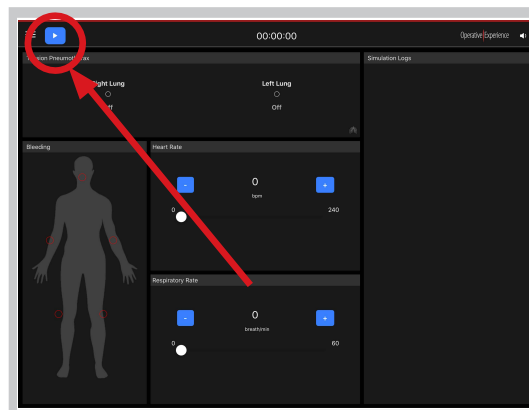


# TCCS Male QUICK START GUIDE

---
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
## 11 Powering Off



+ Stop Simulation by pressing the **Stop** button on the Instructor Interface.

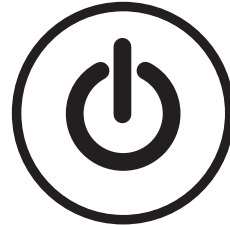

+ Power off simulator by pressing the Power Button in the battery compartment.

The illuminated light flashes until the unit automatically powers off.

### Questions and Support

For in-depth answers on how to use your simulator consult the User Manual downloadable from the OEI App welcome screen or from the website at [operativeexperience.com/support](http://operativeexperience.com/support).

If you need personal assistance with your simulator, contact us at  
Phone: (410) 287-5748  
Email: [support@operativeexperience.com](mailto:support@operativeexperience.com)

