



  
**RealMom™**

---

QUICK START GUIDE

## Table of Contents

Equipment Identification, Downloading the OEI App on the Tablet .....	3
Powering On the Simulator, Joining the Tablet WiFi Network .....	4
Instructor Tablet, Training Mode .....	5
Placing the Baby .....	6
Begin Simulation, Powering Off .....	7
Technical Support and Contact Information.....	8




## 1


## Equipment Identification



+ RealMom & Baby

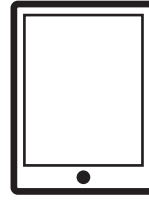

+ Instructor Tablet

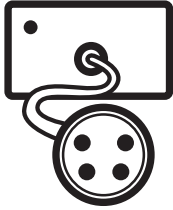

+ Power Supply



+ Patient Monitor Tablet

\*Simulated blood and lubricant also supplied.

## 2

## Downloading the OEI App on the Tablet




# Operative Experience


- + Turn on the tablet and select an available WiFi network
- + Set up tablet with your iTunes or Google Play account
- + Download the OEI App from the iPad or Google App store
- + Go to WiFi settings and “forget” the WiFi network you used to download the App
- + Further help and instructions are available at [operativeexperience.com/support](http://operativeexperience.com/support)

### 3


## Powering On the Simulator

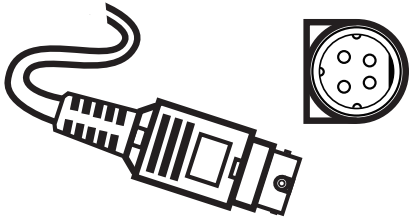

+ Attach Power to Right Shoulder

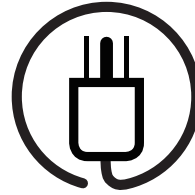

+ Connect the Power to AC

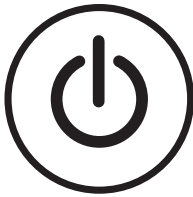

- + Make sure baby is not loaded into the Simulator and turn on the Simulator on the shoulder.
- + A blue light will illuminate
- + Allow Simulator to cycle through the boot up sequence (approx 2 minutes).

### 4

## Joining the Tablet WiFi Network




- + Join the Simulator WiFi on the tablet
- + Your simulator Wi-Fi network is "OVD" followed by its serial number
- + Simulator Wi-Fi password is: oeiaadmin

## 5

## Instructor Tablet

- + Open the OEI Sim Application on the instructor tablet
- + To change language go to Settings in the App menu

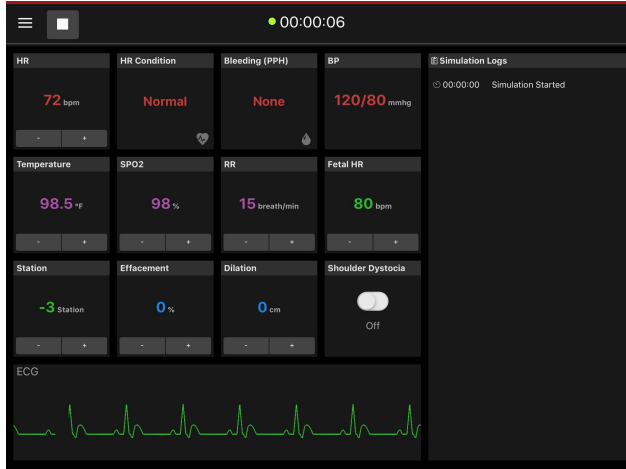

Make sure you are connected to the simulator network and enter the Simulator IP in the field below.




- + If the following message appears on the screen, type the Simulator IP address: 10.42.0.1

## 6

## Trainee Mode


- + The OEI Sim app also has a trainee mode display which displays the simulator physiology such as heart and respiratory rate. Select Trainee mode in the Settings menu

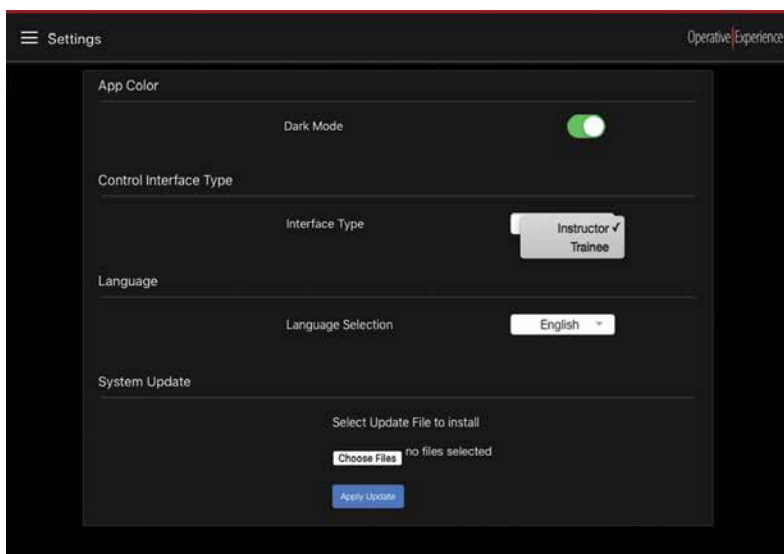

## Placing the Baby



- + Generously coat the baby and placenta with OEI supplied lubricant
- + Open the abdominal and uterine opening and insert baby into cavity

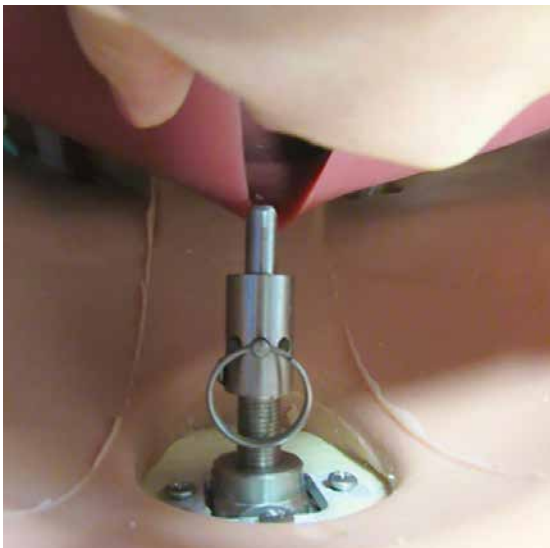


- + Orientate the baby according to scenario
- + Carefully attach the baby to the metal pusher using the cavity in the top or bottom of baby
- + Lubricate the vaginal area inside and outside with lubricant

## 8


## Begin Simulation

- + Press the play button on the Instructor Interface Tablet to begin the simulation
- + Set dilation to 10cm before starting delivery
- + See full user manual at [operativeexperience.com/support](http://operativeexperience.com/support) for detailed instructions

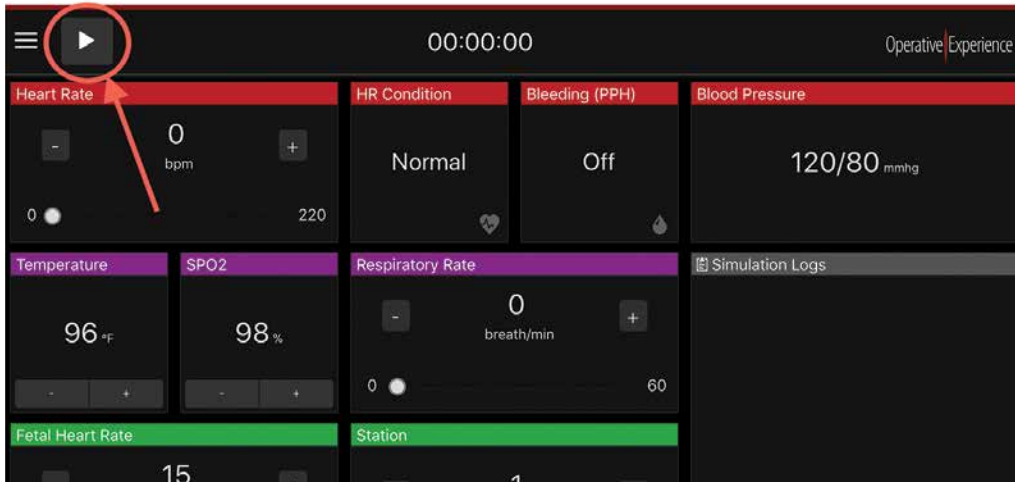

## 9

## Powering Off

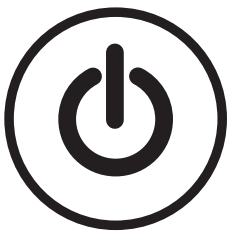

- + Stop Simulation by pressing the Stop button on the Instructor Interface Tablet
- + Power off simulator by pressing ON/OFF switch on the shoulder and holding for four seconds. The illuminated blue light will flash, the unit will power off automatically.

## Questions and Support

Contact us if you have a question  
or need assistance with your simulator.

Phone: (410) 287-5748

Email: [support@operativeexperience.com](mailto:support@operativeexperience.com)

Operative | Experience

[operativeexperience.com](http://operativeexperience.com)