

# TCCS Pro Series Male

## QUICK START GUIDE

To get started, please follow these easy steps:

### 1 Equipment Identification




Simulator



2-Way  
Communication



Hand Pump  
(for blood)

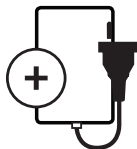

Battery Pack  
and Charger



1 Instructor Tablet  
1 Patient Monitor Tablet

### 2 Download the OEISim Pro App onto the Tablet

- + Turn on the tablet and select an available WiFi network
- + Download the OEISim Pro application from the iOS or iPadOS App store, Mac OS App store, Google Play store, or Microsoft store
- + Go to the tablet's WiFi settings and disconnect the WiFi network you used to download the application



### 3 Charge Battery

- + Plug the battery pack charger into a standard electrical outlet. The red light indicates charging and turns green when the batteries are fully charged.
- + Charge the batteries six (6) to eight (8) hours (or overnight).

